



CRUTCH
R E M E D Y™
COMFORT IN MOTION™



Crutch Remedy™ Cushion & Cover Sets

Directions for Use



**100%
Latex-Free**

30-Day Merchandise Limited Warranty

Remedy Mobility Products (RMP) warrants this product to be free from defects in materials and workmanship.

This warranty is extended only to the original purchaser/user of our products.

RMP warrants its product to be free from defects in material and workmanship for 30 days in unused condition by original purchaser. If within in such warranty period any such product shall be proven to be defective, such product shall be replaced at RMP's option. This warranty does not include any labor or shipping charges incurred in replacement of any such product. RMP's sole obligation shall be limited to providing replacement parts.

To return a product, please call our customer service department at 1-(888)-Why-Remedy - (888) 949-7363 or contact us via email RemedyMobility@FCIUSA.net for return authorization, approval and instructions.

NOTE: You may notice some variations in the gray coloring and/or some small voids or bubbles in your Crutch Remedy Cushions. This is a result of the normal manufacturing process and does not affect the performance or comfort of your Crutch Remedy Cushions in any way.

WARNING

Check cushions and covers for damage with every use. If cushions become torn, ripped or damaged discontinue use. If cushion and/or hand grip covers become loose or damaged discontinue use.

Please consult with your healthcare professional to determine proper height, adjustment and usage of your crutches. For safety instructions and proper crutch adjustment procedures please contact a healthcare professional and/or refer to the directions for use provided with your crutches.

Basic Crutch Safety Checklist:

Located on the sides of the crutches, you may find "snap buttons" which allow for adjustment of the height of your crutches. These snaps must be fully engaged and locked in place before the crutches are put to use; they should protrude through their respective adjustment holes. Check each crutch to ensure they are both adjusted evenly one to the other, and that any wing nuts or screws are tightened and secure.

The crutch cushions have a weight limit of 350 lbs/pair.

Remedy Foam™ Cushions and all Crutch Remedy™ Cover Sets are Latex-Free.



CRUTCH
R E M E D Y™
COMFORT IN MOTION™

Remedy Mobility Products

4669 Means Drive, Plano, Texas 75024

Phone: (888) WHY-REMEDY

(888) 949-7363

www.crutchremedy.com



Remedy Foam™ Cushions and all Crutch Remedy™ Cover Sets are Latex-Free and are 100% manufactured in the USA.

© 2008 CH Partners, LLC.
Crutch Remedy, Remedy Foam and Comfort in Motion
are trademarks of CH Partners, LLC. Patents Pending.

CR-1005 Rev A - 02/2008

**Remedy Foam™ Cushions and all Crutch Remedy™ Cover Sets
are Latex-Free.**

step one

Your current crutches are likely fitted with hard rubber arm pads, which need to be removed prior to replacing them with Crutch Remedy™ Cushions.

Remove Standard Pads

Stretch the rubber pad firmly until you are able to start lifting it off the crutch (Figure 1). Pull up firmly and remove completely (Figure 2). You are now ready to install your Crutch Remedy™ Cushion.

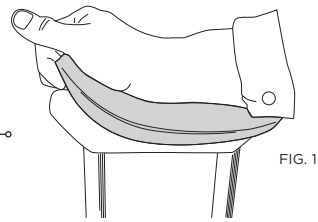


FIG. 1

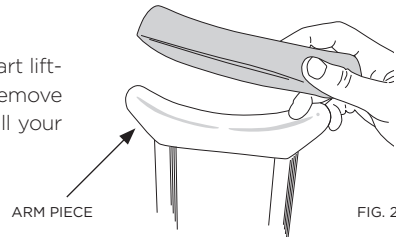


FIG. 2

ARM PIECE

step two

Install Crutch Remedy™ Cushion

Starting on one end of the crutch's arm piece, place the front (higher or taller) end of the Crutch Remedy™ Cushion over one end (Figures 3 & 4), pulling gently to allow enough stretch to fit over the other end. Straighten cushion so that it is evenly and squarely positioned on the arm piece (Figure 5).

NOTE: Both crutch cushions are the same. There is no left or right cushion; however, each cushion has a front and back.

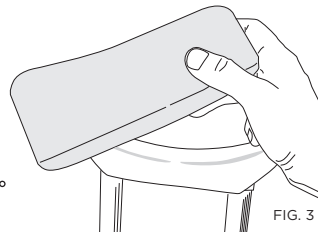


FIG. 3

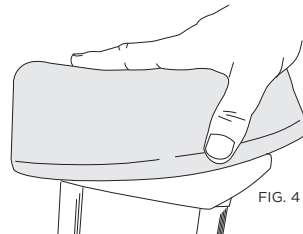


FIG. 4

←← front

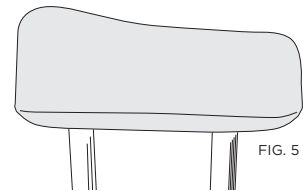


FIG. 5

step three

Install Cushion Covers

Notice the difference in the height from the front end to the back end of your cushion. Your covers are made to fit the same way. Stretch your cover over the cushion (Figure 6), and you're ready to begin using your new cushions (Figure 7).

WASHING COVERS: Machine washable in cold water and dryer safe.

←← front

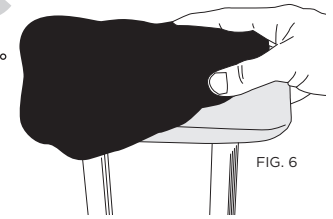


FIG. 6

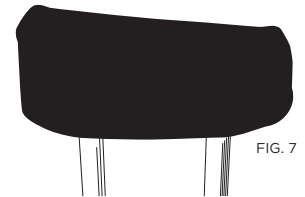


FIG. 7

hand grip

Install Hand Grip Covers

Your matching Hand Grip Covers are easy to apply (Figure 8). Simply wrap around your existing standard, round hand grip and secure with the fastening strip (Figures 9 & 10).

NOTE: Try several positions for the fastening strip, and find out where it is most comfortable for you.

WASHING COVERS: Machine washable in cold water and dryer safe.

⚠ WARNING

Use of the Crutch Remedy™ Cushion Will Raise the Height of Your Crutch approximately two inches (2").

Make sure to re-adjust the height of your crutches downward and your hand grips upward to accommodate this added height. If you are unsure how to adjust your crutches properly, SEEK assistance from a healthcare professional.

If your crutches are already adjusted down as far as possible, or your hand grips are already adjusted up as far as possible, then you will need to seek a replacement in order to properly use Crutch Remedy™ Cushions.

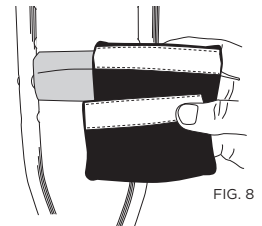


FIG. 8

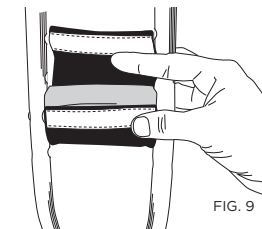


FIG. 9

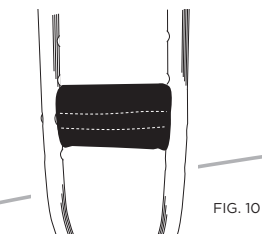


FIG. 10